## Read the instructions carefully before use

**Contents:** 6 individual packages for 6 patients in a box

**Each package includes:**

- 1 syringe of pH neutral stain-removing gel, Powerprep+™, for use exclusively by professionals, 1.2g
- 1 syringe of dental whitening gel, 12% stabilized hydrogen peroxide, 1ml
- 2 application tips
- 1 tube of nourishing and soothing treatment, 5ml
- 1 capsule of prophylactic paste, enriched with amorphous calcium phosphate, 0.40ml

**And, separately**

- 12 application trays
- 6 application tray cases
- or
- 6 pairs of "Boil & Bite" fitted dental trays

**Precautions:**

1. Store in a cool place, away from light and sources of heat.
2. Keep out of reach of children.
3. Do not swallow. If swallowed, consult a doctor.
4. Pregnant or breastfeeding women must consult their physician before use.
5. Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

**Warnings:**

Consumers of alcohol or tobacco should refrain from smoking and/or drinking during the 48-hour period after using NATURAL+™ GentlePower Pro in order to receive the best results.  
Do not apply the NATURAL+™ GentlePower Pro dental whitening gel to teeth with metal amalgams.

**Directions:**

**Preparation of the trays**

- Make unique thermoforming application trays for the patient's use.
- or
- Follow the instructions provided with "Boil & Bite" fitted dental trays.

**Procedure**

1. Prior to the teeth whitening procedure, clean the teeth by using an air powder polisher. If you don't have one, perform descaling or cleaning of the teeth to be treated.
2. Use the trays and fill them with the pH neutral stain-removing gel, Powerprep+™, after positioning the application tip on the syringe. Leave the trays into the mouth for 15 minutes.

3. Remove them, rinse them and reuse them for the teeth whitening procedure.
4. Apply, after positioning the application tip on the syringe, the NATURAL+™ GentlePower Pro dental whitening gel, homogeneously, into the trays.
5. Place the application trays on the teeth and explain to the patient that it is not necessary to firmly close the jaw in order to get a better result.
6. Leave them in the mouth for an hour.
7. Remove the application trays, rinse them, and then, rinse the mouth.
8. Using the hand piece, apply the prophylactic paste, enriched with amorphous calcium phosphate, to the surface of the teeth.
9. Apply the nourishing and soothing treatment to the teeth and gums, massaging steadily, in case of residual sensitivity.
10. Give to the patient what remains of the nourishing and soothing treatment for him/her to use it at home on teeth and gums every hour until the tube is empty.

**Ingredients:**

- pH neutral stain-removing gel, Powerprep+™, for use exclusively by professionals, 1.2g  
Distilled Water, Citric Acid, Potassium Nitrate, Fumed Silica, Pigments
- Dental whitening gel 12% / Activator, 1ml  
Water, Hydrogen Peroxide, Glycerine, Thickener, Potassium Nitrate, Sodium Hydroxide, Sodium Fluoride, Di-sodium Hydrogen Phosphate
- Nourishing and soothing treatment, 5ml  
Aqua, Potassium Nitrate, Sea Salt, Polysorbate 20, Cellulose Gum, Aroma, Sodium Methylparaben, Potassium Sorbate, Dicalcium Phosphate, Eugenol, Eugenia Caryophyllata, Fluoride Sodium, Isoeugenol
- Prophylactic paste, enriched with amorphous calcium phosphate, 0.40ml  
Sodium Fluoride (including Fluoride Ion), FD&C Red#40, FD&C Blue#1

Store in a cool place, away from sunlight

CE 1014

The natural  
**Solution**  
for teeth  
whitening

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