Ecological active agent
Neutral pH

meodental
www.meodental.com

Meodental, F92100
info@meodental.com

CE 1014

NATURALTM

The natural solution for teeth whitening

HOME

Read the instructions carefully before use

Home Kit : 6 tear-off sachets for 6 patients

Each sachet includes:

■ 1 syringe (individually wrapped) of dental whitening gel, 6% stabilized hydrogen peroxide, 7ml

Precautions:

1. Store in a cool place, away from sunlight.
2. Keep out of reach of children.
3. Do not swallow. If swallowed, consult a doctor.
4. Pregnant or breastfeeding women must consult their physician before use.
5. Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

Warning:

Consumers of alcohol and/or tobacco should refrain from smoking and/or drinking as much as possible throughout the duration of the NATURALTM Home dental whitening gel treatment, in order to receive optimum results.

Do not apply the NATURALTM Home dental whitening gel to teeth with metal amalgams.

Directions:

1. Take the application trays given to you by your dental surgeon.
2. Apply the NATURALTM Home dental whitening gel by pressing on the syringe plunger, a dispenser built into the mechanism releases the required quantity for each tooth.
3. Place the application trays on the teeth. It is not necessary to close your jaw more firmly in order to receive a better result.
4. Leave them in the mouth for an hour, even an hour and a half. Repeat this process daily, as instructed by your dentist.
5. Remove the application trays, and then rinse your mouth.
6. Thoroughly rinse the application trays and keep them in a dry, clean place until the next session.

Ingredients:

■ Dental whitening gel / Activator, 7ml:

CE 0297 GDF GmbH, D61191

Gel: Aqua, Hydrogen Peroxide, Silica, PE6-8

Activator: Aqua, Glycerin, Carboxymethylcellulose, Potassium Nitrate, Sodium Hydroxide, Sodium Fluoride, Dead Sea Salt (natural), CI 14720

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