Ecological active agent
Neutral pH

meodental
www.meodental.com

Meodental, F92100
info@meodental.com

CE1014

NATURALTM

The natural solution for teeth whitening

Read the instructions carefully before use

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Contents: 6 individual sachets corresponding to 6 patients

Each sachet includes:

- 1 **twın syringe** of dental whitening gel, 35% stabilized hydrogen peroxide, 2ml
- 1 **twın syringe** of activator (red), 0.5ml

Precautions:

1. Store in a cool place, away from light and sources of heat.
2. Keep out of reach of children.
3. Do not swallow. If swallowed, consult a doctor.
4. Pregnant or breastfeeding women must consult their physician before use.
5. Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

Warning:

Consumers of alcohol or tobacco should refrain from smoking and/or drinking during the 48-hour period after using **NATURAL[®] Professional** in order to receive the best results.

Do not apply the **NATURAL[®] Professional** dental whitening gel to teeth with metal amalgams.

Directions:

1. Follow the instructions attached to the **NATURAL[®] Professional №2** kit for all steps prior to and following the insertion of the dental whitening gel.
2. Remove the caps from the dental whitening gel and activator syringes.
3. Take the Luer Lock connector provided in the **NATURAL[®] Professional №2** kit and screw the two ends of the syringes together with the connector.
4. Press the two syringe plungers alternately (gel + activator) until a homogenous gel is obtained. Subsequently, let the gel rest for about 5-8 minutes in the syringe to obtain an ideal consistency and ready-to-use.
5. Unscrew the syringe containing the mixture and apply the ready-to-use **NATURAL[®] Professional** dental whitening gel uniformly onto the surface of the teeth.
6. Use the lamp to activate the properties of the **NATURAL[®]** dental whitening gel and select the appropriate program, "Intensive" or "Progressive".
7. After completing the dental whitening session, proceed to the teeth cleaning stage by aspirating the **NATURAL[®] Professional** dental whitening gel using the appropriate hand piece connected to the unit. Then, rinse the teeth, using the spray. At the same time, during this last stage, aspirate the surplus water.

For further instructions, refer to the instructions attached to the **NATURAL[®] Professional №2** kit.

Ingredients:

- Dental whitening gel / Activator, 2.5ml:

CE 0297 GDF GmbH, D61191

Gel: Hydrogen Peroxide, PEG, Silica
Activator: Aqua, Thickener, Potassium Nitrate, Sodium Fluoride, Sodium Hydroxide, Dye

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