HOME

The natural Solution for teeth whitening

 **NATURAL⁺** TM 6%

English	2-3
Français	4-5
Deutsch	6-7
Português	8-9
Русский	10-11
Español	12-13
Italiano	14-15

meodental
www.meodental.com

Read the instructions carefully before use

Valid for 1 patient

This kit comes with a detachable sachet containing a syringe (individually wrapped) containing 7ml of dental whitening gel, 6% stabilized hydrogen peroxide, to carry out the complete treatment at home.

- 2 plates for application trays
- 1 application tray case
- 1 toothbrush in a case
- 1 gentle stain-removing toothpaste, **Powerprep+™**, 30ml
- 1 tube of nourishing and soothing treatment, 5ml
- 1 container with dental floss and mirror
- 1 shade guide

Precautions:

Keep out of reach of children.

Avoid contact with eyes. In case of accidental contact, rinse thoroughly.

Directions:

For dentists:

1. With this kit is a detachable sachet containing a syringe (individually wrapped) containing 7ml of **Natural+™ Home** dental whitening gel, 6% stabilized hydrogen peroxide.
2. Make unique thermoforming application trays for the patient's use and put them with the rest of the kit.

For patients:

1. Use the shade guide and mark with a cross the starting color.
2. Take the toothbrush provided in the kit and brush the teeth for 3 minutes using the gentle stain-removing toothpaste, **Powerprep+™**, also provided. Complete this stage by using dental floss and pay particular attention to the spaces in-between the teeth.

3. Take the application trays given to you by your dental surgeon.
4. Apply the **Natural+™ Home** dental whitening gel by pressing on the syringe plunger, a dispenser built into the mechanism releases the required quantity for each tooth.
5. Place the application trays on the teeth. It is not necessary to close your jaw more firmly in order to receive a better result.
6. Leave them in the mouth for an hour, even an hour and a half. Repeat this process daily, as instructed by your dentist.
7. Remove the application trays, and then rinse your mouth.
8. Thoroughly rinse the application trays and keep them in their case, provided in the kit, until the next session.
9. Apply the nourishing and soothing treatment to the teeth and gums, massaging steadily, in case of residual sensitivity.
10. Check the achieved color with the shade guide and compare with the starting color.

Ingredients:

■ Gentle stain-removing toothpaste, **Powerprep+™**, 30ml

Sorbitol 70%, Water (Aqua), Hydrated Silica, Glycerin, Cocamidopropyl Betaine, Sodium Citrate, Sodium Monofluorophosphate, Cellulose Gum, Sodium Bicarbonate, Citric Acid, Titanium Dioxide, Flavor (Supplement), Sodium Saccharin, Sodium Methylparaben, Sodium Propylparaben
Manufactured: CTS, IL

■ Nourishing and soothing treatment, 5ml

Aqua, Potassium Nitrate, Sea Salt, Polysorbate 20, Cellulose Gum, Aroma, Sodium Methylparaben, Potassium Sorbate, Dicalcium Phosphate, Eugenol, Eugenia Caryophyllata, Fluorure Sodium, Isoeugenol
Manufactured: Cosinus, F

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