Professional

The natural Solution for teeth whitening

 **NATURAL+**TM

English	2-3
Français	4-5
Português	6-7
Русский	8-9
Español	10-11
Deutsch	12-13
Italiano	14-15

meodental
www.meodental.com

Read the instructions carefully before use

Home Kit: 6 individual packages for 6 patients

Each box includes:

- **1 syringe** of neutral pH stain-removing gel, for use exclusively by professionals, 1.2g
- **1 syringe** of gingival protection gel, 1.2g
- **1 Luer Lock** connector
- **3 tips**
- **1 tube** of nourishing and soothing treatment, 5ml
- **1 capsule** of prophylactic paste, enriched with Amorphous Calcium Phosphate, 0.40ml

Precautions:

1. Store in a cool place, away from light and sources of heat.
2. Keep out of reach of children.
3. Do not swallow. If swallowed, consult a doctor.
4. Pregnant or breastfeeding women must consult their physician before use.
5. Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

Directions:

1. Clean the teeth by using an air powder polisher. If you don't have one, perform descaling or cleaning of the teeth to be treated, according to the clinical case.
2. Preferably, apply moisturising cream to the contours of the mouth and lips prior to treatment.
3. Position the mouth opener in the patient's mouth.
4. Apply the neutral pH stain-removing gel, for use exclusively by professionals, to all surfaces of the teeth to be treated and leave it for 3 minutes.
5. Rinse, then aspirate the surplus water.
6. Apply the gingival protection gel whilst preparing the dental whitening treatment by setting up the syringe applicator provided in the kit. Ensure, beforehand, that the gums are completely dry.

The use of a gingival protection (dental dam) during any dental whitening treatment in the chair is essential. This helps to isolate the gums and soft tissue around the teeth to be treated.

The dentist must ensure that the tongue and lips do not come into contact with the product (through the use of cheek and labial retractors for example), and that the patient does not suffer from any kind of allergy to any of the ingredients.

With the tip of the syringe, apply the gel along the treatment area, 2 mm wide and 2mm thick, then, polymerize.

7. Apply the **NATURAL+™ Professional** dental whitening gel by following the instructions indicated, which can be found with the syringes in the **NATURAL+™ Professional №1** kit (individual sachets).
8. After completing the dental whitening session, proceed to the teeth cleaning stage by aspirating the **NATURAL+™ Professional** dental whitening gel using the appropriate hand piece connected to the unit. Then, rinse the teeth, using the spray. At the same time, during this last stage, aspirate the surplus water.
Once certain that the patient's teeth are completely cleared of any trace of gel, and that they are completely clean, remove the gingival protection using dental tweezers and then dry the teeth using cotton wool, dabbing carefully.
9. Apply the nourishing and soothing treatment to the teeth and gums, massaging steadily, in case of residual sensitivity.
10. Using the hand piece, apply the prophylactic paste, enriched with Amorphous Calcium Phosphate, to the surface of the teeth.

Ingredients:

- **Neutral pH stain-removing gel, for use exclusively by professionals, 1.2g**

CE Prime Dental Manufacturing Inc, USA

Distilled Water, Citric Acid, Potassium Nitrate, Fumed Silica, Pigments

- **Gingival protection, 1.2g**

CE 0510 Prime Dental Manufacturing Inc, USA

Tri-ethylene Glycol Dimethacrylate, Bisphenol A Diglycidyl Methacrylate, Multifunctional Monomers, Amine, Camphorquinone

- **Nourishing and soothing treatment, 5ml**

Manufactured: Cosinus, F

Aqua, Potassium Nitrate, Sea Salt, Polysorbate 20, Cellulose Gum, Aroma, Sodium Methylparaben, Potassium Sorbate, Dicalcium Phosphate, Eugenol, Eugenia Caryophyllata, Fluoride Sodium, Isoeugenol

- **Prophylactic paste, enriched with Amorphous Calcium Phosphate, 0.40ml**

CE 0510 Premier Dental Products Company, USA

Sodium Fluoride (including Fluoride Ion), FD&C Red#40, FD&C Blue #1

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